

Neater Cup Holder (ND-MH) - Instructions

Description

- The Neater Cup Holder (ND-MH) can be fitted to a wheelchair or other structure to hold the Neater Drinker Metallic (ND-M) or other suitable sized cup securely.
- For maximum flexibility, strong releasable cable ties are used to fix the holder in place.

Set up instructions

- See pictures below.
- Use the cable ties to hold the cup holder in a suitable place. Because they are releasable, you can experiment with different positions. Two ties can be put together to make one longer tie.
- Make sure the holder will not twist around or sag to an inconvenient position. Use additional ties to prevent this.
- You can cut the ties shorter and remove sharp edges to tidy the set-up.

Example set-ups

- Attachment to rear of wheelchair

Cup Holder Releasable Cable Tie



Made in UK by
Neater Solutions Ltd
info@neater.co.uk
Tel: 01298 23882
www.neater.co.uk

Safety

- Drinkers should only be used for drinking consumable liquids.
- They should not be used by people with swallowing difficulties or who are liable to choke, except under the supervision of a trained Speech and Language Therapist.
- Drinks should not be scalding hot.
- Your mouth should always be higher than the mug or siphoning can occur and cause choking.
- Clean all parts before use and check that no cracks have appeared in any components or that any parts can come loose and cause choking.
- Take care not to stick the straw in your eye or into the back of your throat.



- Attachment to wheelchair armrest. (Shown with Neater Drinker Metallic in place)

